

Pet Care Station



3 in 1 Display Solution




Quick Guide Manual

OPCOM Care[®]

OPCOM Pet Care Station

HD video display and COT (Chip-on-the-Tip) design with 4.5mm HD LED distal end & 600mm flexible inserting-tube for all purposes. Pets care, professional of pets care service. Easy to have close views of any part of pet body channel, such as: Nostril, Ear hole, Teeth, Skin, Hair, Fingernail, Lice checking and any portion of the pets where is not easy to look in detail. Particularly for pets clinic in distance diagnosis.

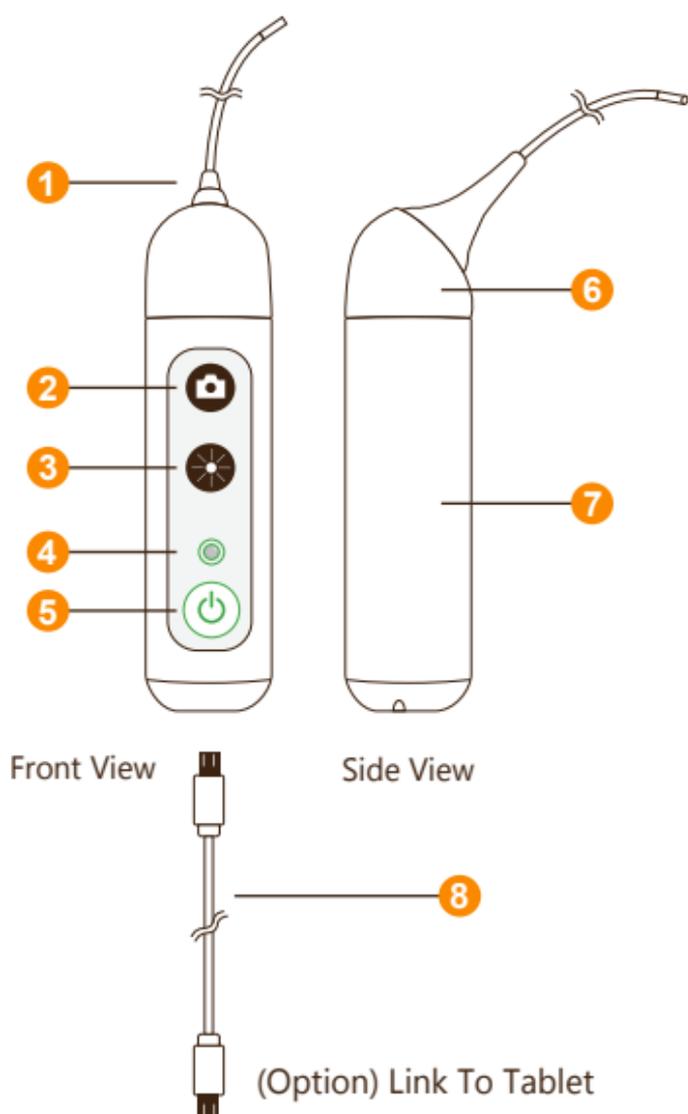



-
- 1. EndoCam Camera** 1.3M Pixels camera.

 - 2. Snapshot button**
 - Press button to take a snapshot.
 - Hold button down for 1-2 seconds to start video recording.

 - 3. Brightness button**
 - Light intensity of 10 segments
 - When you reach the brightest when the lights will flash
 - Go back from 9 to 0 after the cycle

 - 4. LED Indicator**
 - Non-connection with USB cable**
 - Power on :Blinking 45sec. to access into Wi-Fi Mode
 - Sanpshot :Blink once
 - Recording :Blinking continually
 - Stop recording :Stop blinking & back to  on

 - Low Battery: Blinking continually and need recharge immediately
 - EndoCam Camera Head disconnection: Blinks for 5 sec. then powers off automatically (After camera head connection is normal, press the power button and restart again)

 - Connection with USB cable**
 -  Fully Charge (keep charging 1-2 hrs.)

 -  Charging

 - 5. Power ON/OFF Button** Press 3 second to power ON.
Press 1 second to power OFF.

 - 6. EndoCam Camera Head** Camera and LED light module.

 - 7. Battery** Location where rechargeable battery is placed into the unit.

 - 8. USB Cable** Micro USB port at the bottom of the unit.

 - 9. Charge** Support operating time up to 2.5 hours per each full charge.



Operation

1. Download App

To download mobile application to your Mobile device:

- Android: Go to Google play download "OPCOM Care" App.
- iOS: Go to App store download "OPCOM Care" App.

2. Paring Device to Mobile Device :

Power on OPCOM Care unit, and then turn on Wi-Fi on your mobile device.

- Open App, to Menu
- Wi-Fi Pairing, connect to Pet Care Station_xxxx from the Wi-Fi list.

(Option) USB On TO Go (OTG) mode:

1. Turn off the power
2. Plug micro USB at the bottom of OPCOM Care Unit and plug another head of micro USB into tablet.



3. Live View :

After Wi-Fi connected to OPCOM Care unit, back to Home screen, and live-view display on your Mobile or OPCOM Care System Tablet.

4. P-CAM mode :

OPCOM Care unit in power-off state, connect device's micro-USB to PC USB port, then open PC AP which support UVC, the live-view screen will display on PC. You could also download OPCOM AP from OPCOM website.

5. Remark :

Mobile / Application User's Manual available at: www.opcom.com



4.6mm
Camera Head



Microview
Scope 90°



Disposable
Cover



USB Charger



USB
OTG



iOS +
APP

For detailed warranty information, Please visit

www.opcom.com

